# AHP1750

## Indicator Strips

PREvention™

For Use with Accel PREvention Concentrate (7% Diluted 1:40)

Part No. AHP1750

### Description

Accelerated Hydrogen Peroxide® (AHP®) Indicator Strips are easy to use dip-and-read reagent strips to confirm the presence of the minimum recommended concentration of hydrogen peroxide, the active ingredient in AHP disinfectant solutions. **AHP1750 Test Strips were DESIGNED for use with Accel PREvention Concentrate diluted to 1:40 (0.1750% or 1750ppm).**

### Direction for Use

#### Disinfection (1:40)

1. Dip the padded end of the test strip into the diluted solution and remove immediately.
2. At the same time start the timer and shake the strip in a whipping motion three times to prevent excess liquid pooling on the strip.
3. The test strip colour should match the colour block between 35 and 40 seconds to indicate a **PASS**.

#### Sanitizing (1:128)

1. Dip the padded end of the test strip into the diluted solution and remove immediately.
2. At the same time start the timer and shake the strip in a whipping motion three times to prevent excess liquid pooling on the strip.
3. The test strip colour should match the colour block between 110 to 120 seconds (1 minute 50 seconds to 2 minutes) to indicate a **PASS**.

*NOTE: Delayed reading of the strip colour may invalidate the results. The strip colour will continue to darken after the read time.*

The following table provides an easy comparison of the dilution rates for the AHP products to parts per million (ppm).

Accel PREvention Concentrate Dilution Rate	PPM
Disinfection 1:40	1750 ppm
Sanitizing 1:128	500 ppm

### Storage

Store the AHP Indicator Test Strips at room temperature between 15°C–30°C (59°F–86°F). Do not expose the strip to direct sunlight. Ensure the cap is closed tightly after each use.

### Important Precautions

Keep the strip in the original container. Remove only the required strip before testing and immediately replace the cap tightly. Exposure of the uncovered strip to light and moisture may cause discoloration of the strip. Do not use the strip after the expiration date (opened or unopened). Do not touch the reagent pad with finger or any other surface. Once opened, the test kits have a shelf life of 6 months. Unopened test kits have a 24-month shelf life.

### Product Availability

AHP1750 Indicator Strips are packaged 50 strips per bottle, one (1) bottle per package. Part Number AHP1750 when ordering.

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**Virox Technologies Inc.**  
2770 Coventry Rd., Oakville, ON L6H 6R1  
1-800-387-7578 | www.virox.com

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