

PREempt[®] WIPES

One-Step Disinfectant
Cleaner Wipes formulated
with **AHP**[®]
Patented



Wipes

WHY CHOOSE PREempt™?

- > **CLEANER** Effective disinfectant as well as a cleaner resulting in added confidence that disinfection can occur.
- > **FASTER** Rapid and realistic contact times on a broad spectrum of difficult to kill pathogens.
- > **RESPONSIBLE** Designed to be easier on surfaces, staff and occupants. Users are not exposed to VOCs (volatile organic compounds) and PPE** is not required as per health and safety rating.
- > **SUSTAINABLE** The active ingredient, Hydrogen Peroxide, breaks down into water and oxygen leaving no active residues which helps to reduce environmental impact.

DIRECTIONS FOR USE

For Use as a One-Step Disinfectant Cleaner Product:

1. Pre-clean heavily soiled areas.
2. Pull towelette from canister and wipe hard, non-porous environmental surfaces.
3. **All surfaces must remain visibly wet for 1 minute.** Use a 5 minute contact time for TB[†]. Use a 10 minute contact time for fungi.
4. Allow to air dry.

For Use as a Non-Food Contact Sanitizer:

1. Pre-clean heavily soiled areas.
2. Pull towelette from canister and wipe hard, non-porous environmental surfaces.
3. Allow the surface to remain wet for 30 seconds.
4. Wipe surfaces dry.
5. Change towelette after each use.

For use in laboratories, clean rooms and other critical environments that require cleaning and surface disinfection. This includes work stations, fume hoods, laboratory counter tops, equipment and other hard non-porous environmental surfaces.

AVAILABLE PRODUCTS

PREempt™ One-Step Disinfectant Cleaner Wipes

EPA REG. NO: 74559-3

SHELF LIFE: 3 Years

CONCENTRATION OF ACTIVE:
0.5% Hydrogen Peroxide

- **Format Size:** 6"x7" – 160ct Wipes
Product Code: 21221
Packaging Specifications: 12x1 Canister
- **Format Size:** 7.5" x 10" – 10ct Wipes (Soft Pack)
Product Code: 21223
Packaging Specifications: 1 Case x 6 Inner Cartons x 20 Soft Packs

EFFECTIVE AGAINST A BROAD-SPECTRUM OF BACTERIA, *VIRUSES AND FUNGI IN 1 MINUTE!

*Virucidal: 1 Min.

- *Poliovirus Type 1
- *Kills HIV-1 (AIDS virus)
- *Kills Norovirus (Feline Calicivirus as the surrogate)
- *Human Coronavirus
- This product has demonstrated effectiveness against Influenza A virus and is expected to inactivate all Influenza A viruses including Pandemic 2009 H1N1 Influenza A virus.

Bactericidal: 1 Min.

- *Pseudomonas aeruginosa*
- *Staphylococcus aureus*
- *Salmonella enterica* (formerly known as *Salmonella choleraesuis*)

Antibiotic-Resistant Bactericidal Activity:

- Methicillin-resistant *Staphylococcus Aureus* (MRSA)
- Vancomycin-resistant *Enterococcus faecalis* (VRE)
- *Escherichia coli* with Extended Spectrum Beta-lactamase resistance (ESBL)

Fungicidal: 10 Min.

- *Trichophyton mentagrophytes*

Tuberculocidal: 5 Min.

- [†]*Mycobacterium bovis* (BCG)

Broad-Spectrum Non-Food Contact Sanitizing: 30 Sec.

- *Klebsiella pneumoniae*
- *Salmonella enterica* (formerly known as *Salmonella choleraesuis*)
- *Pseudomonas aeruginosa*
- *Staphylococcus aureus*
- *Escherichia coli* O157:H7



PREempt™ One-Step Disinfectant Cleaner Wipes are made of non-woven, meltblown polypropylene wipe material, ensuring even surface coverage and increasing the ability and efficiency of AHP® for fine particle removal.

*Please refer to reference sheet for a complete list of pathogenic organisms, additional features, and claims.

PREempt™ branded products are manufactured by Virox Technologies Inc.

Accelerated Hydrogen Peroxide® (AHP®) and PREempt™ are registered trademarks of Virox Technologies Inc.



For more information on this or our other products, visit:

> virox.com

2770 Coventry Road, Oakville, ON L6H 6R1 CANADA
1-800-387-7578 | www.virox.com



Engineering Revolutionary Disinfectants for the War Against Microbes