

PREempt®

Ready to Use Sporicidal Disinfectant
Cleaner formulated with



Sporicidal

- **CLEANER** Effective disinfectant as well as a cleaner resulting in added confidence that disinfection can occur.
- **FASTER** Rapid and realistic contact times on a broad spectrum of difficult to kill pathogens.
- **RESPONSIBLE** Designed to be easier on surfaces, staff and occupants. Users are not exposed to VOCs (volatile organic compounds).
- **SUSTAINABLE** The active ingredient, Hydrogen Peroxide, breaks down into water and oxygen leaving no active residues which helps to reduce environmental impact.

DIRECTIONS FOR USE

Sporicidal Disinfection of Hard Non-Porous Surfaces:

1. Apply liquid to surfaces using a cloth or disposable wipe.
2. To achieve Sporicidal disinfection ensure surface remains wet for 10 minutes.
3. Thoroughly rinse to ensure all product residue has been removed.

Note: This product is NOT to be used as a terminal Sterilant/High Level Disinfectant for critical instruments and medical devices. This product is intended for use as an Environmental Surface Sporicide on hard non-porous surfaces in *C. difficile* patient rooms.

Cleaning Surfaces Contaminated with Bodily Fluids:

Appropriate Personal Protection must be worn during all cleaning and disinfection procedures.

1. Remove excess body fluids with absorbent materials.
2. Apply liquid to surfaces using a cloth or disposable wipe and let stand for 1 minute.
Disinfect contaminated area.

A SURFACE DISINFECTANT EFFECTIVE AGAINST

Sporicidal: 10 Minutes.

- *Clostridium difficile* [Clinical isolate]
- *Clostridium sporogenes* (ATCC 7955)
- *Bacillus subtilis* (ATCC 19659)

Designed for cleaning and sporicidal disinfection of hard non-porous surfaces.

AVAILABLE PRODUCTS

PREempt™ Sporicidal Liquid

DIN: 02451891

SHELF LIFE: 2 Years

CONCENTRATION OF ACTIVE:
4.5% Hydrogen Peroxide

- **Format Size:** 1 L Bottle
- **Product Code:** 11701
- **Packaging Specifications:** 12x1 L

PREempt™ branded products are manufactured by Virox Technologies Inc.

Accelerated Hydrogen Peroxide® (AHP®) and PREempt™ are registered trademarks of Virox Technologies Inc.

For more information on this or our other products, visit:

➤ virox.com 

2770 Coventry Road, Oakville, ON L6H 6R1 CANADA
1-800-387-7578 | www.virox.com



Engineering Revolutionary Disinfectants for the War Against Microbes