

# PREempt™ CS20 Tool & Implement Sterilant

Sterilizes in 20 minutes with no dilution required  
and reusable for up to 14 days.

## POWERFUL

Sterilizes tools in only 20 minutes. Health Canada registered sterilant and high level disinfectant.

## COMPATIBLE

With your professional stainless steel and hard plastic tools and implements.



## CONVENIENT

No dilution required.  
Re-use for up to 14 days  
in a soaking tray.<sup>1</sup>

## RESPONSIBLE

Biodegradable. Active ingredient breaks down into water and oxygen.

<sup>1</sup>High level disinfection is recommended for semi-critical tools or implements, that come in contact with mucous membranes or non-intact skin during use, but do not ordinarily penetrate the skin or enter normally sterile areas of the body. In personal service settings, this can include tools such as reusable foot files, tweezers used for ingrown hairs, microblade handles, nail clippers and cuticle pushers, or needle holders. Health Canada is responsible for oversight of high-level disinfectant label claims and has approved the re-use of PREempt™ CS20 for 14 days. Exceptions to this allowance occur if the solution is visibly soiled, or if the indicator strip fails to produce the required measurement. In these cases, the solution should be discarded even if the 14-day period has not yet been reached.

For more information visit: [ViroxProBeauty.ca](http://ViroxProBeauty.ca)

# PREempt<sup>®</sup> CS20

Powered by AHP<sup>®</sup>  
**ACCELERATED<sup>®</sup>**  
HYDROGEN PEROXIDE  
Patented

## DISINFECTION PROTOCOL

**1 CLEAN/RINSE/DRY**  
Clean tools using water and detergent. Rinse and dry thoroughly.



**2 FILL SOAKER**  
Pour PREempt<sup>™</sup> CS20 directly into soaking tray. Do not dilute.



**3 SOAK TOOLS & RE-USE SOLUTION**  
Fully immerse tools in PREempt<sup>™</sup> CS20 for **20 minutes**. Re-use solution for up to 14 days.\*



\*The solution must be discarded after 14 days or sooner as dictated by AHP7 Test Strips. Read label for full list of claims and use-directions.

**4 RINSE, DRY & STORE**  
Rinse tools with water and dry manually using clean cloth or disposable towel. Store in a clean container marked "DISINFECTED".



### BEST PRACTICES

- Tool disinfection should take place after each and every client service to prevent the spread of germs.
- Only soak your tools for 20 minutes. Do not leave tools soaking overnight.
- Use only high quality stainless steel and hard plastic professional tools to avoid compatibility issues.†
- Let your clients know that the tools you are using have been properly cleaned and disinfected.
- Disinfect only reusable tools and ensure all single-use tools are properly disposed.

### FOR USE WITH

- Nail Clippers
- Nippers
- Cuticle Pushers
- Comedone Extractors
- Reusable Foot Files
- Reusable Nail Files
- Drill Bits (Excl. carbide)
- Tweezers

† Manufacturers that provide *Instructions For Use (IFUs)* with their tools and implements should be given preference.

For more information visit: [ViroxProBeauty.ca](http://ViroxProBeauty.ca)

Warning: You must read and follow the instructions on the label for our product and use the product for the particular applications specified on the product label. To the full extent permitted under applicable law, Virox Technologies Inc. shall not be liable for any damages whatsoever, arising out of or otherwise in connection with any failure to follow those instructions or any use of its products which is not specified on the product's label. The instructions on the product label are not a warranty, express or implied, and Virox Technologies, Inc. expressly disclaims any implied warranties, including any warranty of fitness for a particular purpose. Under applicable law, some of the above waivers or disclaimers of warranty, or exclusions of liability for certain types of damages may not apply to you. In such cases, our liability will be limited to the fullest extent permitted by applicable law.