

# PREempt®

Surface and Instrument  
Cleaners and Disinfectants  
formulated with



## WHY CHOOSE PREempt™?



➤ **CLEANER** Effective disinfectant as well as a cleaner resulting in added confidence that disinfection can occur.



➤ **FASTER** Rapid and realistic contact times on a broad spectrum of difficult to kill pathogens.



➤ **RESPONSIBLE** Designed to be easier on surfaces, staff and occupants. Users are not exposed to VOCs (volatile organic compounds) and PPE\*\* is not required as per HMIS rating.



➤ **SUSTAINABLE** The active ingredient, Hydrogen Peroxide, breaks down into water and oxygen leaving no active residual which helps to reduce environmental impact.

Accelerated Hydrogen Peroxide® (AHP®) is a patented synergistic blend of commonly used, safe ingredients that when combined with low levels of hydrogen peroxide dramatically increases its germicidal potency and cleaning performance without sacrificing user safety or environmental sustainability.

PREempt™ products provide a unique balance between safety and efficacy addressing varied infection prevention and control needs including environmental surface cleaners and disinfectants, instrument cleaners, high level disinfectants and sterilants.

## SURFACE PRODUCTS

### PREempt™ Wipes

- For disinfection and cleaning of work stations, equipment, and other hard non-porous environmental surfaces.
- PREempt™ RTU has easy to use formats with no mixing or measuring required.
- Made of low linting non-woven, meltblown polypropylene wipe material, ensuring even surface coverage and increasing the ability and efficiency of AHP® for fine particle removal.

*PREempt™ cleaners and disinfectants are based on the patented Accelerated Hydrogen Peroxide® (AHP®) technology and were designed with your key decision making criteria in mind: Cleaning Performance, Germicidal Efficacy, Health & Safety, and Environmental Sustainability.*



For more information on these products, visit:

➤ [virox.com](http://virox.com)

## PREempt™ Ready To Use

- For use in lab areas that require environmental cleaning and surface disinfection such as work stations, equipment and other hard non-porous environmental surfaces.
- PREempt™ RTU has easy to use formats with no mixing or measuring required.

## PREempt™ Concentrate

- For cleaning and disinfection of large environmental and hard non-porous surfaces in laboratories, clean rooms and other critical environments.
- PREempt™ Concentrate In-Use Solution is designed to be easier on employees and staff therefore requiring no Personal Protective Equipment (PPE) as per HMIS rating.

	WIPES	READY TO USE LIQUID	CONCENTRATE
Virucidal	3 Minutes	3 Minutes	5 Minutes (1:40 dilution)
Bactericidal	3 Minutes	3 Minutes	5 Minutes (1:40 dilution)
Fungicidal	3 Minutes	3 Minutes	5 Minutes (1:40 dilution)
Tuberculocidal	3 Minutes	3 Minutes	5 Minutes (1:40 dilution)
Sanitizer	30 Seconds	30 Seconds	30 Seconds (1:128 dilution)
Shelf Life	2 Years	2 Years	2 Years, 30 Days Diluted
Certifications	n/a	EcoLogo/UL Approved	EcoLogo/UL Approved

## HIGH LEVEL DISINFECTANTS, STERILANTS AND INSTRUMENT CLEANERS

### PREempt™ CS 20 (Ready to Use)

- Chemical sterilant and high level disinfectant reprocessing of critical and semi-critical devices and instruments.
- Contains no aldehydes or VOCs (volatile organic compounds) and requires no special ventilation.

### PREempt™ HLD5 (Ready to Use)

- High level disinfectant and chemical sterilant for reprocessing critical and semi-critical devices and instruments.
- Contains no aldehydes or VOCs (volatile organic compounds) and requires no special ventilation.

### PREempt™ WASH (Concentrate)

- Instrument and cart cleaner for use in ultrasonic and automatic washing machines.
- 3 minutes for heavy duty cleaning and ultrasonic applications (1:64 dilution), 5 minutes for manual cleaning.
- Reduces biofilm and debris and has an exceptional cleaning profile against carbohydrates, lipids and proteins. Requires no additional heat or special equipment, and contains no enzymes or VOCs (volatile organic compounds) and is bacteriostatic.

	CS 20	HLD5
High Level Disinfection	20 Minutes	5 Minutes
Chemical Sterilant	20 Minutes	6 Hours
Shelf Life	1 Year, 14 Day Re-use Claim	1 Year, 14 Day Re-use Claim

## AVAILABLE PRODUCTS

### PREempt™ Wipes

**DIN:** 02436671

- Format Size:** 6" x 7" – 160ct Wipes  
**Product Code:** 11221  
**Packaging Specifications:** 12 x 1 Case
- Format Size:** 7.5" x 10" – 10ct Wipes (Soft Pack)  
**Product Code:** 11223  
**Packaging Specification:** 1 Case x 6 Inner Cartons x 20 Soft Packs

### PREempt™ Ready to Use

**DIN:** 02436698

- Format Size:** 1 L Bottle  
**Product Code:** 11101  
**Packaging Specifications:** 12 x 1 L
- Format Size:** 4 L Bottle  
**Product Code:** 11105  
**Packaging Specifications:** 4 x 4 L

### PREempt™ Concentrate

**DIN:** 2436701

- Format Size:** 4 L Bottle  
**Product Code:** 11305  
**Packaging Specifications:** 4 x 4 L
- Format Size:** 200 L Drum  
**Product Code:** 11311  
**Packaging Specifications:** 1 x 200 L

### PREempt™ CS 20 (Ready To Use)

**DIN:** 20437295

- Format Size:** 1 L Bottle  
**Product Code:** 11401  
**Packaging Specifications:** 12 x 1 L
- Format Size:** 4 L Bottle  
**Product Code:** 11405  
**Packaging Specifications:** 4 x 4 L

### PREempt™ HLD5 (Ready To Use)

**DIN:** 20437287

- Format Size:** 4 L Bottle  
**Product Code:** 11505  
**Packaging Specifications:** 4 x 4 L

### PREempt™ WASH (Concentrate)

- Format Size:** 4 L Bottle  
**Product Code:** 11605  
**Packaging Specifications:** 4 x 4 L

\* Please refer to reference sheet for a complete list of pathogenic organisms, additional features, and claims.

PREempt™ brand products are manufactured by Virox Technologies Inc.

Accelerated Hydrogen Peroxide® and PREempt™ are registered trademarks of Virox Technologies Inc.

2770 Coventry Road, Oakville, ON CANADA L6H 6R1  
1-800-387-7578 | www.virox.com



Engineering Revolutionary Disinfectants for the War Against Microbes