

PREempt[®]

High Level Disinfectant and Sterilant for Medical Devices and Instruments formulated

with **ACCELERATED[®]**
HYDROGEN PEROXIDE
Patented



HLD5

WHY CHOOSE PREempt™?

- **FASTER** As a Ready to Use solution, there is no need to mix extra chemicals or add an activator. Rapid high level disinfection results in faster application, which saves time and money.
- **RESPONSIBLE** Requires no special ventilation, is aldehyde free, and has no volatile organic compounds (VOCs). It is a more responsible solution for disinfection without the use of harmful chemicals.
- **SUSTAINABLE** The active ingredient, Hydrogen Peroxide breaks down into water and oxygen, leaving no active residual and reducing environmental impact.
- **COMPATIBLE** Tested to ensure compatibility that preserves your investments in medical devices and instruments.

DIRECTIONS FOR USE

Cleaning / Decontamination:

Blood and other body fluids must be thoroughly cleaned from hard, nonporous surfaces of medical devices and instruments before application of the disinfectant or sterilant. For complete disinfection or sterilization of medical instruments and equipment, thoroughly clean, rinse and dry objects before immersion in PREempt™ HLD5 solution.

High Level Disinfection:

Place pre-cleaned, rinsed and dried medical devices and instruments in solution of undiluted PREempt™ HLD5. Allow devices to soak for at least 5 minutes. Rinse with sterile water or potable tap water. PREempt™ HLD5 may not be suitable for all instruments. (SEE NOTICE TO USER on the product label).

Sterilization:

Place pre-cleaned, rinsed and dried medical devices and instruments in solution of undiluted PREempt™ HLD5. Allow devices to soak for at least 6 hours. Remove and rinse with sterile water. PREempt™ HLD5 may not be suitable for all instruments. (SEE NOTICE TO USER on the product label).

PREempt™ HLD5 is a high level disinfectant and sterilant for processing heat sensitive semi-critical and critical medical devices for which heat sterilization is not suitable.

AVAILABLE PRODUCTS

PREempt™ HLD5

LICENCE NO. 103056

SHELF LIFE: 12 Months

RE-USE CLAIM: 14 Days

CONCENTRATION OF ACTIVE:
2% Hydrogen Peroxide

1. Format Size: 4 L Bottle
Product Code: 11505
Packaging Specifications: 4x4 L

5-MINUTE HIGH LEVEL DISINFECTION 14 DAY REUSABLE SOLUTION!

Virucidal: 5 Min.

- Poliovirus, Sabin strain type 1 (ATCC VR-192), which allows for a Broad-Spectrum Virucide claim against most enveloped and non-enveloped viruses.

Bactericidal: 5 Min.

- *Pseudomonas aeruginosa*
- *Staphylococcus aureus*
- *Salmonella enterica*

Fungicidal: 5 Min.


- *Trichophyton mentagrophytes*

Tuberculocidal: 5 Min.

- *Mycobacterium terrae*

Sporicidal: 6 Hrs.

- *Bacillus subtilis*
- *Clostridium sporogenes*

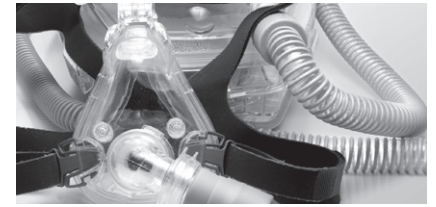

PREempt™ HLD5 is powered by the patented Accelerated Hydrogen Peroxide® (AHP®) Technology. AHP is a synergistic blend of commonly used, safe ingredients that when combined with hydrogen peroxide forms a powerful disinfectant. This ensures that high level disinfection and sterilization can be achieved in realistic contact times while offering unparalleled user safety.

*Please refer to reference sheet for a complete list of pathogenic organisms, additional features, and claims.

PREempt™ brand products are manufactured by Virox Technologies Inc.

Accelerated Hydrogen Peroxide® and PREempt™ are registered trademarks of Virox Technologies Inc.



For more information on this or our other products, visit:

> virox.com

2770 Coventry Road, Oakville, ON CANADA L6H 6R1
1-800-387-7578 | www.virox.com

