

# PREempt<sup>®</sup>

## Instrument Cleaning Solution Specially Formulated for Use with Esthetic Tools



Wash

### WHY CHOOSE PREempt™?

- **CLEANER** PREempt™ WASH is bacteriostatic and has exceptional cleaning efficiency against carbohydrates, lipids and proteins. It prevents biofilm and organic debris build up, reducing maintenance activities.
- **FASTER** It is a 3 to 5 minute cleaner with multiple applications. It requires no additional heat or special equipment, accelerating cleaning efficiency while saving time and money.
- **RESPONSIBLE** Enzyme free, does not release noxious fumes, and contains no volatile organic compounds (VOCs). Users are not exposed to harmful chemicals, allowing for a safer cleaning experience.
- **SUSTAINABLE** The active ingredient, Hydrogen Peroxide breaks down into water and oxygen, leaving no active residual and reducing environmental impact.

Enzyme Free | Bacteriostatic | Low Foaming | Heat Not Required

### DIRECTIONS FOR USE

#### Use Dilutions:

- 1:128 – Manual Cleaning (3-5 Min.)
- 1:256 – Manual Cleaning (5 Min.)

PREempt™ WASH is a very versatile formulation that has the ability to clean all soil types present on tools and instruments without the use of enzymes and has the added benefit of providing bacteriostatic properties by helping to control the bacterial growth.

### AVAILABLE PRODUCTS

PREempt™ WASH

**RE-USE CLAIM:** 14 Days

**CONCENTRATION OF ACTIVE:**  
3% Stabilized Hydrogen Peroxide

**Format Size:** 4 L Bottle

**Product Code:** 11605

**Packaging Specifications:** 4x4 L

## DIRECTIONS FOR USE

### Manual Cleaning (1:128 to 1:256 dilution):

1. Dilute PREempt™ WASH with warm water (do not use hot water).
2. Place instrument into solution for 5 minutes using agitation when required.
3. Remove instrument, rinse, inspect for cleanliness and proceed with disinfecting step.
4. Change solution daily or as needed.



*PREempt™ WASH uses the Stabilized Hydrogen Peroxide technology which is a synergistic blend of commonly used, safe ingredients and hydrogen peroxide that ensures you are achieving a superior cleaning performance in realistic contact times while offering unparalleled user safety.*

\*Please refer to reference sheet for a complete list of pathogenic organisms, additional features, and claims.

PREempt™ brand products are manufactured by Virox Technologies Inc.

Accelerated Hydrogen Peroxide® and PREempt™ are registered trademarks of Virox Technologies Inc.



For more information on this or our other products, visit:

**> viroxprobeauty.com**

2770 Coventry Road, Oakville, ON CANADA L6H 6R1  
1-800-387-7578 | [www.viroxprobeauty.com](http://www.viroxprobeauty.com)



*Engineering Revolutionary Disinfectants for the War Against Microbes*